

## Kit Contents



MOLECULAR



# Drug Chemistry Testing Kit Instructions

## Submitting Powder, Plant, Solids

- Use provided spoon to scoop a small amount of bulk powder into powder container (kit item 4)
- Stamp bags can be placed in envelopes (kit item 5)
- MolecularDx will not accept bulk materials (e.g. "Bricks")

## Submitting Sharps

- Use the tube labeled "sharps container" (kit item 2) to place any needle, blades, or sharp material for residue analysis

## Submitting Paraphernalia

- Place any large items into the plastic bag (kit item 6) or directly into the box
- If the item is larger than the box please contact MolecularDx (814-361-6997)

## Seal, Label, and Submit!

- Seal each container of evidence with security tape (kit item 7) and label with identifying marks, initials, date/time
- Place all evidence in plastic bag (kit item 6), seal with security tape and label with initials and date/time
- Include complete requisition and any additional documentation as needed.
- Call MolecularDx for transport options (814-361-6997)

- ① Kit Box, requisition, and instructions (1)
- ② Sharps submission container (1)
- ③ Large Gloves (1)
- ④ Powder submission containers with spoons (2)
- ⑤ Envelopes (2)
- ⑥ Plastic bag (1)
- ⑦ Security tape (5)

